BioSonic Ultrasonic Insert

EN

Instructions

During all scaling procedures, do not contact the lips, cheek and tongue with the activated tip. Avoid gouging the tooth, but rather debride the surface with the side of the activated tip using a wiping motion. **Never scale without adequate water flow, and always keep the activated tip in motion on the tooth surface.**

Cleaning and sterilization

Remove gross debris by placing the inserts in an ultrasonic cleaner basket and immersing them in a **non-ionic, non-ammoniated** cleaning solution. (Recommended solutions: BioSonic UC30 or UC32.) Operate the ultrasonic cleaner in accordance with the manufacturer's directions. Remove inserts from the cleaner and dry completely with an air syringe or clean paper towel. Alternatively, inserts may be carefully hand-scrubbed (using a nylon bristle brush) with detergent and cold water to remove debris, then rinsed and dried. Take extra care to thoroughly rinse all cannulations, using a syringe if necessary to rinse the insert tip.

Use only steam autoclaving to sterilize BioSonic inserts. Sterilize inserts in the same manner as other, similar inserts, using American Dental Association sterilization recommendations. Do not expose the inserts to a temperature above 275°F/135°C.

Do not use cold sterilization, chemical disinfection, or other methods of sterilization, including chemical vapor and dry heat since these methods may severely damage the insert and void the warranty.

NOTE:

- Do not use chemical disinfectants prior to sterilization or rapid deterioration of the insert materials may occur.
- Lubricate the O-ring with water before placing the inserts into the handpiece. Depress the foot pedal to fill the handpiece with water and slowly turn the insert until it snaps into place. Alternatively, lubricating the O-ring with a small amount of petroleum jelly could prolong the life of the O-ring.
- **For optimum performance and increased insert life, use low to medium power settings only.**

Limited Warranty

BioSonic ultrasonic inserts are carefully manufactured and have met stringent quality assurance requirements. If, in

normal use, any insert is found to be defective in material or workmanship within a period of ninety (90) days from the date of original purchase, it will be replaced or, at COLTENE's option, the purchase price will be refunded.

THE LIMITED WARRANTY STATED HEREIN IS THE SOLE AND EXCLUSIVE WARRANTY WITH RESPECT TO THE INSERT, AND COLTENE HEREBY DISCLAIMS ANY AND ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING, WITHOUT LIMITATION, ANY STATUTORY WARRANTIES OR ANY WARRANTIES OF MERCHANTABILITY, FITNESS FOR USE OR FITNESS FOR ANY PARTICULAR PURPOSE.

COLTENE WILL NOT BE RESPONSIBLE FOR ANY INCONVENIENCE, LOSS, INJURY, OR DIRECT, INDIRECT, TORT, EXEMPLARY, PUNITIVE, SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGE ARISING FROM THE POSSESSION OR USE OF THE INSERT. IN NO EVENT WILL COLTENE/WHALEDENT'S LIABILITY EXCEED THE PURCHASE PRICE OF THE INSERT.

Tampering, abuse, misuse, neglect, alteration, accidental damage, failure to follow manufacturer's instructions, improper disinfection or sterilization procedures, bending, re-grinding or re-shaping, or lack of reasonable care with respect to the insert will void this warranty. Use of the insert by anyone other than a licensed dental practitioner or qualified dental hygienist will void this warranty.

COLTENE will not be responsible for damage in transit, and any claim for such damage should be made promptly against the carrier.

In the event of a warranty claim, it is the responsibility of the user to return the defective insert(s) to:

Coltène/Whaledent Inc.
235 Ascot Parkway
Cuyahoga Falls, Ohio 44223/US

Shipping charges are to be paid by the sender (shipper). The return must be accompanied by a written description of the claimed defect. In addition, a warranty claim must be accompanied by a dated sales receipt or other appropriate documentation showing the date of purchase.

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